
Instructions for ML308 Halogen & ML308-LED Examination Lamps

- Remove lamp & 4 black fixing screws from carton.
- If using our ML307 Wall Bracket, attach wall bracket to wall then fix lamp to top of bracket with the 4 screws provided (See diagram below).
- If using our ML309 Mobile Trolley Bracket, attach lamp to top of bracket with 4 screws provided and then place lamp/bracket assembly into top of pole (See diagram below). Tighten thumbscrew on pole if provided.
- If using our ML130 Complete Stand, please read installation instructions supplied with the stand. Then attach lamp to the bracket on top of pole.
- If using our ML305 Bi-Fold Wall Bracket, please read installation instructions supplied with the bracket.

